

RAL[®] Stainer

The RAL Stainer is a compact, efficient and automated bath staining unit. The built-in RFID system helps eliminate protocol errors. The calibrated RAL stains and our robust mechanics result in a high-quality, standardized and re-producible staining. The RAL Stainer together with its associated RAL staining kits are a flexible solution for laboratories that handle routine analyses. The pre-programmed, customizable staining protocols have been optimized in collaboration with reference laboratories.




Features

Intelligent
Pre-programmed,
customisable protocols


Efficient
Low reagent consumption

Safe
Sealed staining vessels


Simple
Ready-to-use reagents

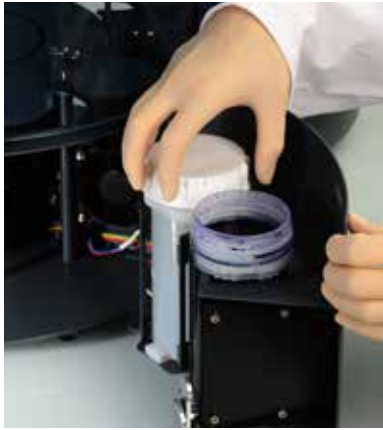


Ready-to-use reagents

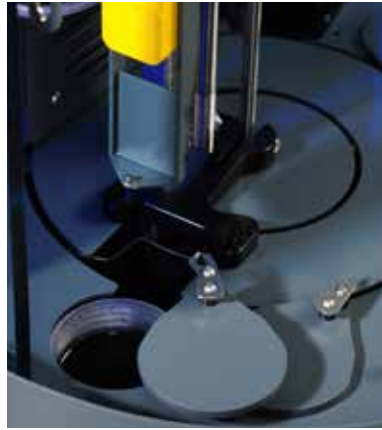


Sealed staining vessels

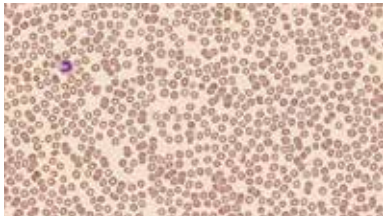


Kit RAL Stainer MCDh - Peripheral blood smear

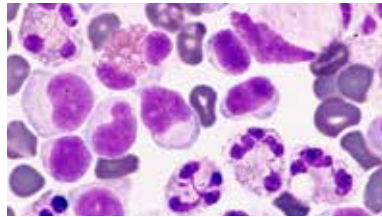

Kit RAL Stainer 555 - Pleural Fluid

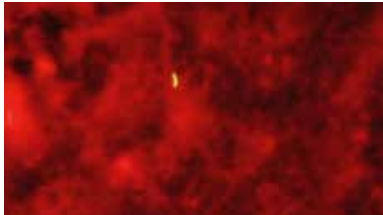


Kit RAL Stainer Fluo-RAL - Mycobacterium tuberculosis

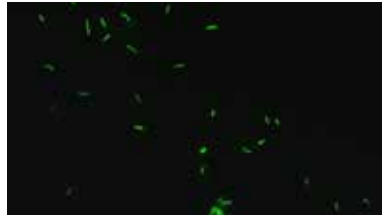

Kit RAL Stainer Fluo-RAL Methylene Blue - Positive sample of Mycobacterium



Kit RAL Stainer Gram-Nicolle - Bacillus Cereus sporulé

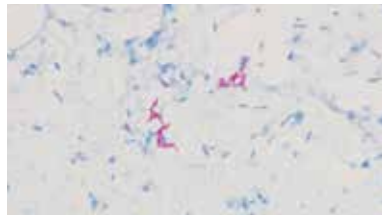

Kit RAL Stainer Cold ZN - Positive sample of Mycobacterium tuberculosis

Available Staining Kits

KIT RAL STAINER MCDH™

- Ref. 360200-0000 | 500 slides



Kit RAL Stainer MCDH™



Kit RAL Stainer 555

KIT RAL STAINER 555

- Ref. 360250-0000 | 300 slides

KIT RAL STAINER FLUO-RAL

- Ref. 360230-0000 | 300 slides



Kit RAL Stainer Fluo-RAL



Kit RAL Stainer Fluo-RAL Methylene Blue

KIT RAL STAINER FLUO RAL METHYLENE BLUE

- Ref. 360270-0000 | 300 slides

KIT RAL STAINER GRAM-NICOLLE

- Ref. 360220-0000 | 500 slides



Kit RAL Stainer Gram-Nicolle



Kit RAL Stainer Gram-Hücker

KIT RAL STAINER GRAM-HÜCKER

- Ref. 360210-0000 | 500 slides

KIT RAL STAINER COLD ZN

- Ref. 360240-0000 | 500 slides



Kit RAL Stainer Cold ZN

RAL® Stainer

Technical Specifications

DIMENSIONS

- Height: 58,7 cm
- Diameter: 40 cm

SUPPLY VOLTAGE RANGE

- 110-230V / 50-60HZ / 150 VA

WEIGHT

- 28 kg / 61,7 lbs

FUNCTIONS

- 2 loading / unloading / drying zones
- 5 stations, (4 staining stations & 1 rinsing station)
- 2x10L containers for rinsing liquid & waste
- Automatic waste draining in 10L
- Number of tests possible: 300 to 500 slides, see IFU for more details
- Time: from 32 sec to 27 min, see IFU for more details
- Life span: 2 weeks after opening

Specifications are subject to change without notice. This product may not be available in all markets.