


CellaVision® Proficiency Software

CellaVision Proficiency Software is a web-based solution designed to support laboratory teams in developing and maintaining expertise in cell morphology.

It enables laboratory managers to create structured learning and assessment that strengthens staff competency over time, while keeping the workflow simple and efficient.

The software can be used on its own or together with CellaVision analyzers and supporting applications, providing a flexible platform for continuous professional development. By combining training, evaluation, and feedback in one environment, it helps laboratories build confidence, improve consistency, and support the delivery of accurate, standardized results.

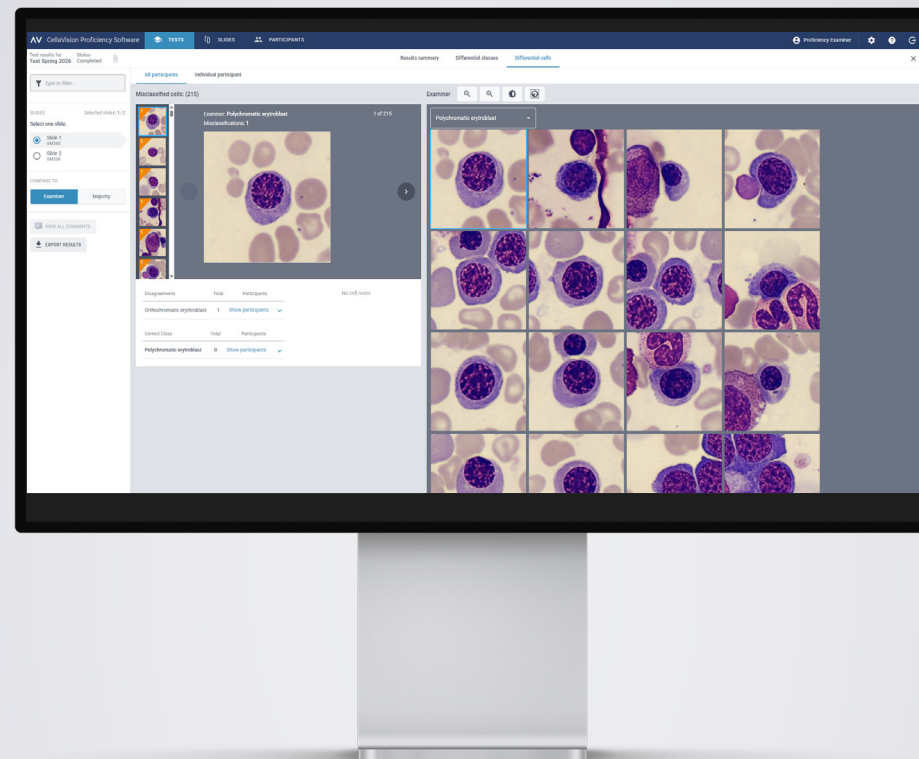

Features

Supports peripheral blood, body fluid, bone marrow aspirate and VET slides

Web-based, with instant access from anywhere

Clear result summary with zoomed overview and search functionality

Results for participants down to a single-cell level

Intuitive navigation between slides, tests, and participants


Automated reporting and notifications for examiners and participants

Adapt slides for training and testing purposes

All slide data is **anonymized**


How does it work?

- The software is designed to train and evaluate staff members' ability to perform peripheral blood, body fluid and bone marrow aspirate differentials
- The web-based tool can be accessed from any internet-connected computer via secure, personalized user logins
- The examiner can create test scenarios by uploading digital slides, either from their own CellaVision database or from a CellaVision-provided slide library
- Participants are automatically notified when a new test is available and prompted to perform differentials on one or more digital slides
- Upon completion, participants can review their results and compare their differentials with those of the examiner and their peers
- The examiner is automatically provided with a performance report highlighting areas for further training and competency development

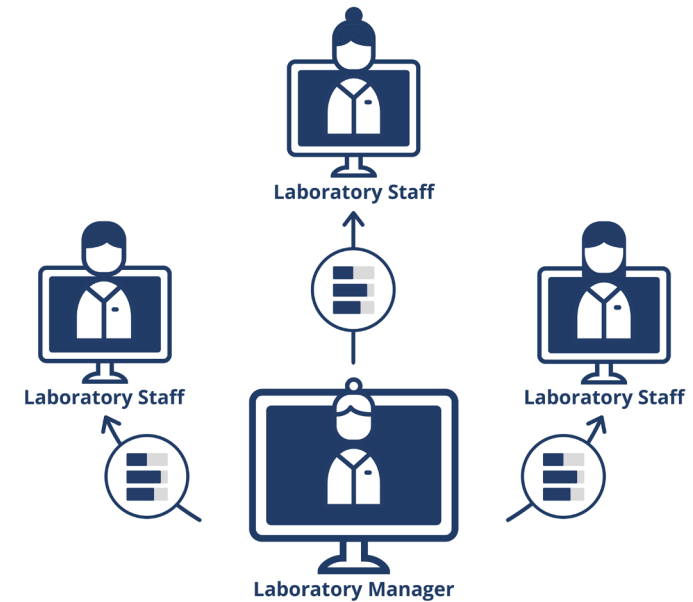


Why CellaVision Proficiency Software?

- Suitable for organizations of all sizes
- Can be tailored to address identified competency gaps and support specific training needs and goals
- Tests are quick and easy to create and manage
- Tests are simple for participants to access and complete
- Results are automatically compiled

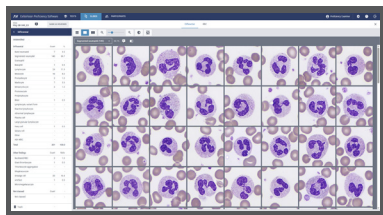

Single WBC gallery view

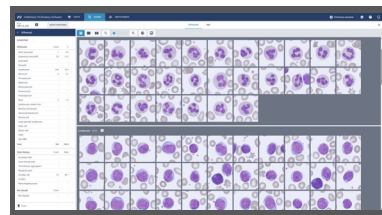

WBC all galleries view

CellaVision® Proficiency Software

Technical Specifications

RECOMMENDED PC SPECIFICATIONS

- Browser: Latest version of Google Chrome, Microsoft Edge (Chromium based), Mozilla Firefox, Apple Safari
- Minimum monitor resolution: 1920 x 1080

DEPENDENCIES

- Transfer tool for users with CellaVision Software version 7.1 or below

LICENCES

- Choose from a range of license options:
 - 10, 30, 50, 70 or 100 users
 - 3, 5 or 7 years

Specifications are subject to change without notice. This product may not be available in all markets.